



Product Reference Guide

Flexible IV Container

Opening/Preparing

Overwrap:

The overwrap acts as a moisture and dust barrier. Water can permeate from inside the container into the overwrap but not in amounts sufficient to affect the solution significantly.



1. Tear the outer wrap at the notch.
 - > Do not remove from overwrap until ready for use.
 - > Remove container from overwrap and discard wrap.



2. Squeeze and visually inspect container for leaks, particulate matter, and discoloration.
 - > If leaks are found, discard unit as sterility may be impaired. Contact ICU Medical's Global Complaint Management at 1 (800) 441-4100.
 - > Some opacity of the plastic due to moisture absorption during the sterilization process may be observed. This is normal and does not affect the solution quality or safety. The opacity will diminish gradually.

Preparation for Administration: (Use aseptic technique)



Close flow control clamp of administration set.



Remove cover from outlet port at bottom of container. Grasp tab and remove with a "peeling" motion to prevent tab from tearing.



Insert piercing pin of administration set into port with a twisting motion until the set is firmly seated (insert piercing pin fully). See full directions on administration set package.



Suspend container from hanger.

 **WARNING:** Do not use flexible container in series connections.