



**RE: Use of TriOx™ Central Venous Oximetry Catheter in Power Infusion Procedures**

Dear Clinical Staff;

Thank you for your interest in ICU Medical Central Venous Oximetry Catheters (CVOC). The CVOC catheter has been evaluated for power infusion procedures to measure maximum flow and pressure capabilities. In this study the distal 16G lumen was selected and Omnipaque® (iohexol) Injection 240 was infused at a rate of 10mL per second. The 16G Lumen was then tested to a minimum hydrostatic pressure of 300psig, where no burst failures occurred.

Thank you again for your interest, if there are further questions or concerns please visit our website at [www.icumed.com](http://www.icumed.com), or contact the corporate offices at 949-366-2183.

Best regards,

A handwritten signature in black ink, appearing to read "Alison Burcar", with a long horizontal line extending to the right.

Alison Burcar  
Vice President of Product Development  
ICU Medical, Inc.