



**RE: GENIE™ Compatibility with the solvent N,N-Dimethylacetamide commonly found in Busulfex / Busulfan.**

Thank you for your interest in the **GENIE™** Closed Vial Access device. The **GENIE™** is a closed, needlefree vial access device that automatically equalizes vial pressure that serves to reduce to a minimum aerosols, vapors and leaks. Closed needlefree devices such as the **GENIE™** are recommended by NIOSH, The National Institute for Occupational Safety and Health, for the safe preparation of hazardous drugs.

It should be noted that the **GENIE™** contains components made of Medical Grade Polycarbonate which is not compatible with a solvent (N, N-Dimethylacetamide) that is used in certain drug preparations such as Busulfex / Busulfan. This is clearly indicated on the Busulfex (Busulfan) package insert. Drug and solvent preparations are continuously changing therefore package inserts should always be reviewed prior to use for compatibility information.

Thank you again for your interest, if there are further questions or concerns please visit our website at [www.icumed.com](http://www.icumed.com) , or contact the corporate offices at (949) 366-2183 or (800) 824-7890.

Technical Services

ICU Medical, Inc.

.