

CLAVE[®] Needleless Connector



Remember to **PRIME**
your CLAVE Connector
by inverting the device
to expel air.



Remember to **SWAB**
your CLAVE Connector
with desired disinfectant
in accordance with
facility protocol.



Remember to
FLUSH your
CLAVE Connector
after each use in
accordance with
facility protocol.

hi


San Clemente, CA USA
(949) 366-2183 (800) 824-7890
www.icumed.com

US. PAT. NOS: 5,685,866; 5,690,612; 5,694,686; 5,695,466; 5,873,862; 5,901,942; 5,928,204; 6,132,403; 6,572,592; 6,599,273; 5,738,663; 6,019,748; 6,325,782; 5,700,248;
6,132,404; 6,669,673; 6,635,044; 6,572,592; 6,682,509; 6,726,672; 6,758,833 Other Patents Pending

CLAVE[®] NeedleFree Connector


Catalog Nos. C1000, C1000G, C1000-GR, C1000-RR, C1000-YR
For Needleless Access to Central, Peripheral, and Arterial Lines

- ➔ Flows more than 185 mL per minute on average at 36" head height.*
- ➔ Holds 60 PSI backpressure.
- ➔ Ultra low residual volume of 0.06cc (deadspace).
- ➔ Compatible with IV solutions, most chemotherapy drugs, and lipids.
- ➔ Contains no natural rubber latex.

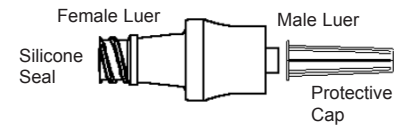

DIRECTIONS FOR USE


1. Using aseptic technique remove CLAVE Connector from package. Do not contaminate.
2. Attach a priming syringe or administration set to female luer of CLAVE and prime in accordance with facility protocol. Invert device to expel air then remove priming syringe.
3. Remove protective cap and attach male luer of CLAVE to extension set or IV catheter.

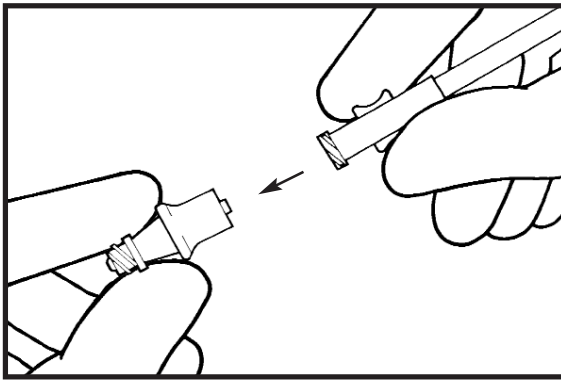


4. To access CLAVE Connector swab silicone seal in accordance with facility protocol.

CAUTION: Do not use needles to access CLAVE Connector.

5. Attach a fully primed syringe or administration set to CLAVE. Grasp CLAVE Connector and firmly push and twist male adapter into CLAVE until tight.

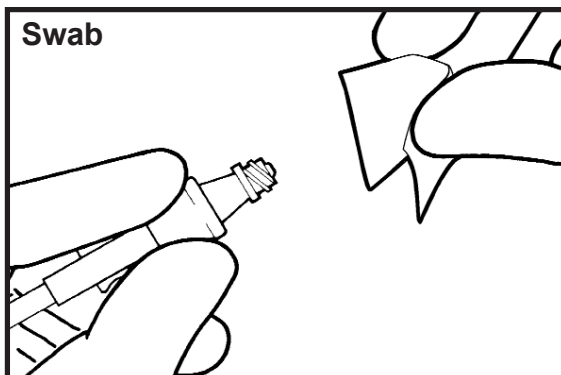


NOTE: If using a rotating luer device, first push and twist luer slip into CLAVE until tight, then lock down the rotating collar. This will ensure a secure connection and optimal flow rates.

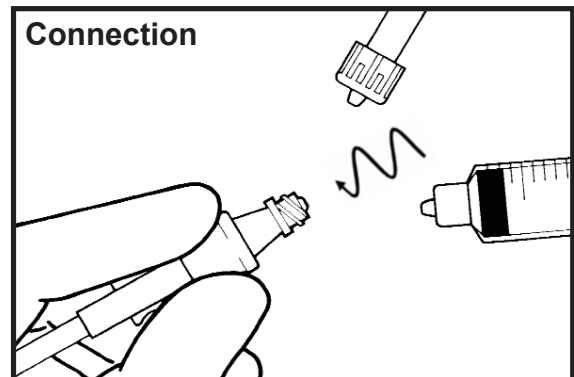

6. To disconnect, grasp CLAVE Connector and twist syringe or administration set away from CLAVE until loose.

NOTE: If using a rotating luer device grasp CLAVE and first unscrew the rotating collar. Next push luer slip into CLAVE to break seal and then twist away from CLAVE until loose.

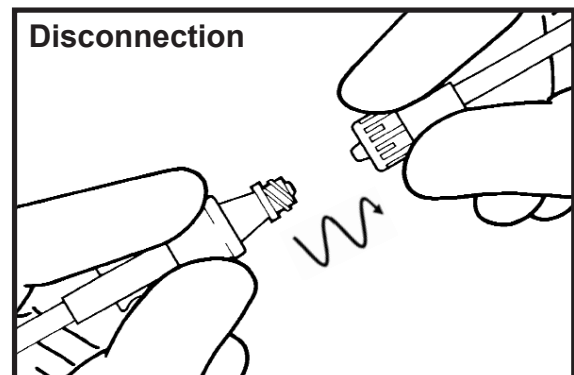

7. Flush CLAVE Connector after each use in accordance with facility protocol.
8. Do not cap CLAVE Connector, device is closed. For subsequent connections repeat from step four (4).

CAUTIONS: Federal (USA) law restricts this device to sale by or on the order of a physician.
It is recommended this device be changed per CDC guidelines or per validated facility protocol
*Mean average flow rate based on Special Evaluation 20.00206 8/97.

hi

San Clemente, CA USA
(949) 366-2183 (800) 824-7890
www.icumed.com

US. PAT. NOS; 5,685,866; 5,690,612; 5,694,686; 5,695,466; 5,873,862; 5,901,942; 5,928,204; 6,132,403; 6,572,592; 6,599,273; 5,738,663;
6,019,748; 6,325,782; 5,700,248; 6,132,404; 6,669,673; 6,635,044; 6,572,592; 6,682,509; 6,726,672; 6,758,833 Other Patents Pending

M1-1049 4/04