

## Spinning ChemoLock Adapter (CL2000S) / ChemoLock Adapter (CL2000)

Attaching an adapter to Administration set.



### To connect a Spinning ChemoLock Adapter with Administration set

- › Grasp administration set collar and twist the Spinning ChemoLock adapter (CL2000S) female luer straight onto administration set luer until a click is heard and it rotates freely. The Spinning ChemoLock adapter is now permanently attached to the administration set.
- › **Note:** If using a traveling spin collar with backstop, pull collar away from tip of luer, push ChemoLock adapter straight onto Luer to make the connection and then lock down the spin collar.
- › A ChemoLock adapter (CL2000) can be attached similarly but will not click on attachment or rotate upon connection.