




Reference Guide for Neutron™

Needlefree Catheter Patency Device

Directions for Use

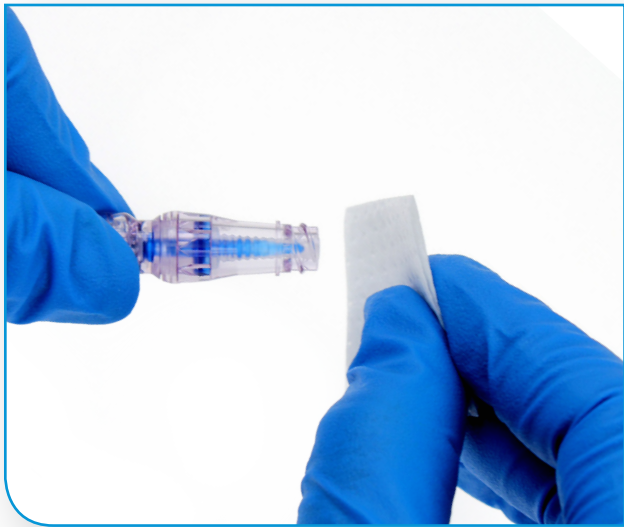

Disinfect to Protect

- › When placing a new Neutron on a catheter, disinfect the catheter hub and prime the Neutron.
- › Before accessing a Neutron, always disinfect the injection site with the approved antiseptic per facility protocol.
- › Scrub the injection site in accordance with facility protocol for appropriate scrubbing and dry times.



Administer or Aspirate

- › Attach IV tubing, syringe or blood tube holder to Neutron by inserting the luer and twisting 1/4 turn, or until a friction fit is achieved.
- › When aspirating blood using evacuated tubes, the blood tube will fill slightly slower because of the Neutron anti-reflux valve.
- › To disconnect, grasp Neutron and then twist the mating luer away from Neutron until loose. Do not hold catheter hub during disconnect as this may cause accidental removal of Neutron from hub.



Flush After Each Use

- › Always flush Neutron with normal saline or in accordance with facility protocol after each use.
- › Use routine flushing in accordance with facility protocol in order to maintain catheter patency.
- › Change Neutron in accordance with facility protocol and CDC guidelines.

Functional Attributes

- › Lipid and Blood Compatible
- › Radiographic Imaging Compatible (CT Compatible)
- › Contains No Latex, Phthalate's (DEHP) or Metal Components, MRI Compatible



Neutron's saline flush option allows you to reduce risks, cost and time associated with Heparin use.

icumedical
human connections