




Reference Guide to Properly Using the NanoClave® Manifold Multi-Port Infusion System

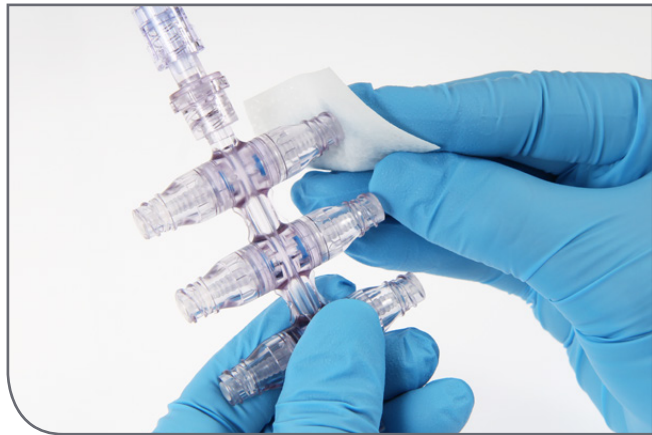

Disinfect to Protect

- › Before accessing each NanoClave, always disinfect the injection site with the approved antiseptic per facility protocol.
- › Scrub the injection site in accordance with facility protocol for appropriate scrubbing and dry times.



Administer

- › Attach IV tubing or syringe to NanoClave by inserting the luer and twisting until secure.
- › Do not over-tighten a luer beyond the friction fit as this may damage both the luer and the NanoClave.
- › To disconnect, grasp NanoClave and then twist mating luer away from NanoClave until loose.



Flush After Each Use

- › Flush the NanoClave with normal saline or in accordance with facility protocol after each use.
- › Use routine flushing in accordance with facility protocol in order to maintain catheter patency.

Functional Attributes

- › Lipid and Blood Compatible
- › Radiographic Imaging Compatible (CT Compatible)
- › Contains No Latex, Phthalates (DEHP) or Metal Components, MRI Compatible



NanoClave's saline flush option can eliminate the risk of Heparin Induced Thrombocytopenia (HIT).