

Use of the Antimicrobial Clave[®] and Antimicrobial MicroClave[®] Connector with MRI

The antimicrobial agent that is used in the Antimicrobial Clave and Antimicrobial MicroClave products is a silver ion, not a silver metal. The silver ion has no ferromagnetic activity and, therefore, will not have a magnetic or heat related reaction within the MRI chamber or procedure. Furthermore, studies on products like wound care dressings, which use the same type of silver ions, have confirmed that these types of silver ions do not cause a disruption in the image itself.

If there are further questions or concerns, please visit our website at www.icumed.com or contact the corporate offices at 949-366-2183 or 800-824-7890.

Technical Services
ICU Medical Inc.