



Product Reference Guide

SwabCap™

Disinfecting Cap for Needlefree Connectors



Peel

- › Remove the protective cover by pulling the tab.
- › Do not remove the SwabCap disinfecting cap from the white plastic holder.



Push and Twist

- › Hold the needlefree connector in one hand while pushing and twisting the SwabCap disinfecting cap in a clockwise direction onto the needlefree connector.



Protect

- › After the SwabCap disinfecting cap has been twisted into place, gently pull the white plastic holder off the needlefree connector.



To Remove for Access to the Valve

- › Two-handed removal must be used to make sure the needlefree connector is not removed unintentionally.
- › Grasp the needlefree connector and twist the SwabCap disinfecting cap counterclockwise away from the connector. Remove and discard.
- › Inspect the needlefree connector to verify that it's connected securely to the catheter.
- › The connector is ready for access. No further swabbing is necessary after removal, but please refer to your hospital policy.

Precautions: Do not use if package is damaged or opened prior to use. Do not freeze. Do not use if product is dry of alcohol. Use by expiration date on package. Two-handed removal must be used to make sure needlefree connector is not removed unintentionally. Single-use only.
Note: Intended to be used and handled by clinical personnel only. Care should be exercised to prevent unintended handling by children.



ANTT® Evaluated Technology promotes safe and effective Aseptic Non-Touch Technique principles by design.

ANTT is a registered trademark of The Association of Safe Aseptic Practice (The-ASAP)

icumedical
human connections