

Spinning ChemoLock Adapter (CL2000S) / ChemoLock Adapter (CL2000)

Attaching an adapter to empty syringe.



To connect a Spinning ChemoLock Adapter with a syringe

- › Attach a Spinning ChemoLock adapter (CL2000S) to a syringe by twisting together until a click is heard and it begins to rotate freely. The Spinning ChemoLock adapter is now permanently attached to the syringe.
- › **Note:** Do not forcibly push a ChemoLock adapter onto a syringe and avoid bending forces at the connection as this may damage the syringe threads and cause an improper connection.
- › A ChemoLock adapter (CL2000) can be attached similarly but will not click on attachment or rotate upon connection.