



# CADD-Solis VIP™

## Ambulatory Home Infusion System

Multi-therapy smart infusion system easy for everyday use

**icumedical**  
human connections

## Designed to be easy to use and enable safe use at home


The CADD-Solis VIP ambulatory infusion pump is ideal for use at home with a small, portable design and easy-to-use interface. The intuitive help screens enable troubleshooting at home and can limit the need for clinician intervention. Colour-coded screens provide instant status of therapy progress, while CADD reservoirs help to support medication security and free-flow protection of fluid delivery. With five versatile delivery modes, the CADD-Solis VIP ambulatory pump helps you provide infusion care from the comfort of home.



## Help increase patient independence

### Alarm help screens


On-pump alarm help screens assist with troubleshooting to help resolve alarm conditions with guided, on-screen instructions. These screens can help reduce clinician intervention and field visits and maintain infusion continuity.

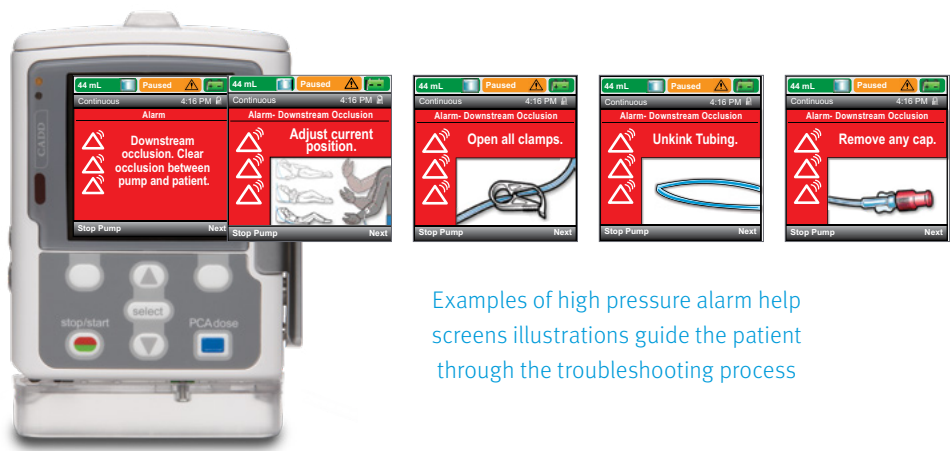


Examples of high pressure alarm help screens illustrations guide the patient through the troubleshooting process

### Differentiated alarms


Differentiated alarms by colour and sound with high (red), medium (amber), and low (blue) priority

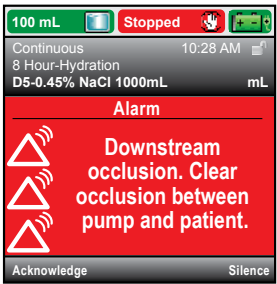

High-priority alarm

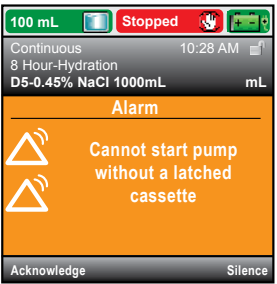


Medium-priority alarm

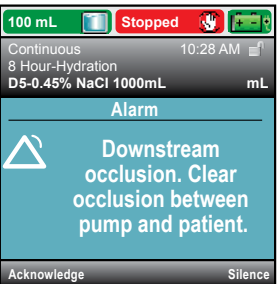

Low-priority alarm

## Intuitive delivery status screens show infusion progress

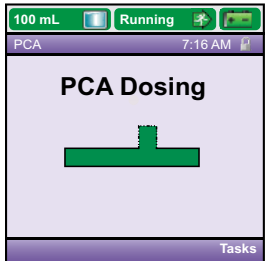


### PCA

Pain management using the following delivery methods individually or in combination with each other: Continuous rate, PCA dose, and Clinician bolus




### Taper

Plateau rate with optional tapering at the beginning and end

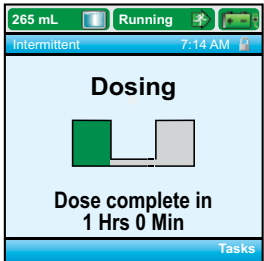


### Intermittent

Specific volume at a scheduled interval over a specified duration

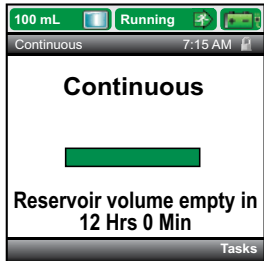

### Continuous

Constant, programmed rate

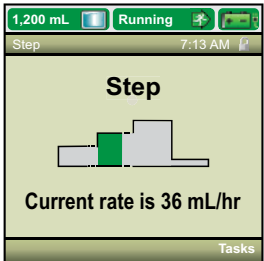


### Step


Incremental rate increases from an initial rate to a specified maximum plateau rate for a total volume

### On-pump reports

On-pump reports provide trending data and help with patient assessments

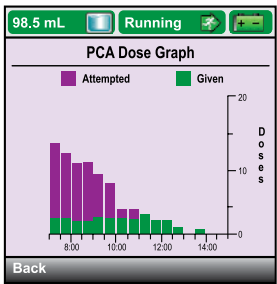

Dosing report screen

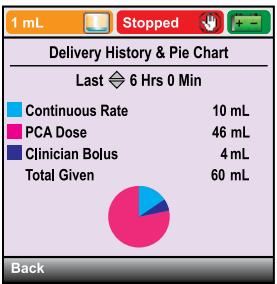

Delivery history report screen

The versatile CADD-Solis VIP ambulatory pump is indicated for intravenous, intra-arterial, subcutaneous, intraperitoneal, perineural, surgical site, epidural space, or subarachnoid space infusions, with five delivery modes.



# CADD-Solis VIP Ambulatory Infusion System

## Easier for Patients and Nurses



Support your patients' independence today.

Call +44 (0)845 850 0445 or visit [www.icumed.com](http://www.icumed.com) to learn more.

The product complies with current legislation and has the corresponding CE marking.  
For additional information, warnings and / or safety precautions, refer to the manufacturer's user manual.

**icumedical**  
human connections