



68.2

CADD-Solis VIP™

Ambulatory Home Infusion System

Multi-therapy smart infusion system easy for everyday use


icumedical
human connections

Designed to be easy to use and enable safe use at home

The CADD-Solis VIP ambulatory infusion pump is ideal for use at home with a small, portable design and easy-to-use interface. The intuitive help screens enable troubleshooting at home and can limit the need for clinician intervention. Colour-coded screens provide instant status of therapy progress, while CADD™ reservoirs help to support medication security and free-flow protection of fluid delivery. With 5 versatile delivery modes, the CADD-Solis VIP ambulatory pump helps you provide infusion care from the comfort of home.




Intuitive delivery status screens show infusion progress

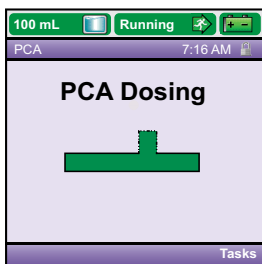


PCA

Pain management using the following delivery methods individually or in combination with each other: continuous rate, PCA dose, and clinician bolus.

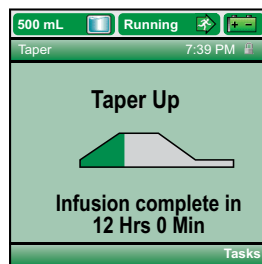


Taper

Plateau rate with optional tapering at the beginning and end

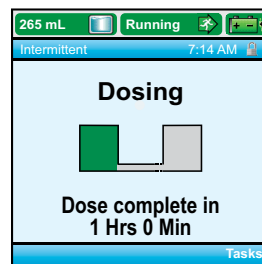


Intermittent

Specific volume at a scheduled interval over a specified duration

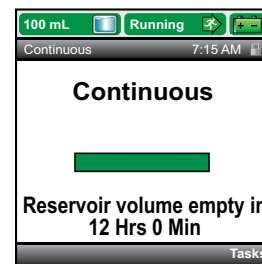

Continuous

Constant, programmed rate

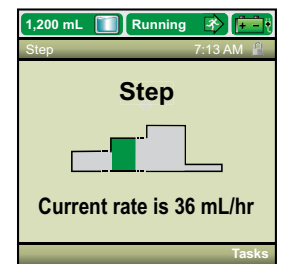

Step


Incremental rate increases from an initial rate to a specified maximum plateau rate for a total volume

The versatile CADD-Solis VIP ambulatory pump is indicated for intravenous, intra-arterial, subcutaneous, intraperitoneal, perineural, surgical site, epidural space or subarachnoid space infusions, with 5 delivery modes.

Help increase patient independence

Alarm help screens


On-pump alarm help screens assist with troubleshooting to help resolve alarm conditions with guided, on-screen instructions. These screens can help reduce clinician intervention and field visits and maintain infusion continuity.

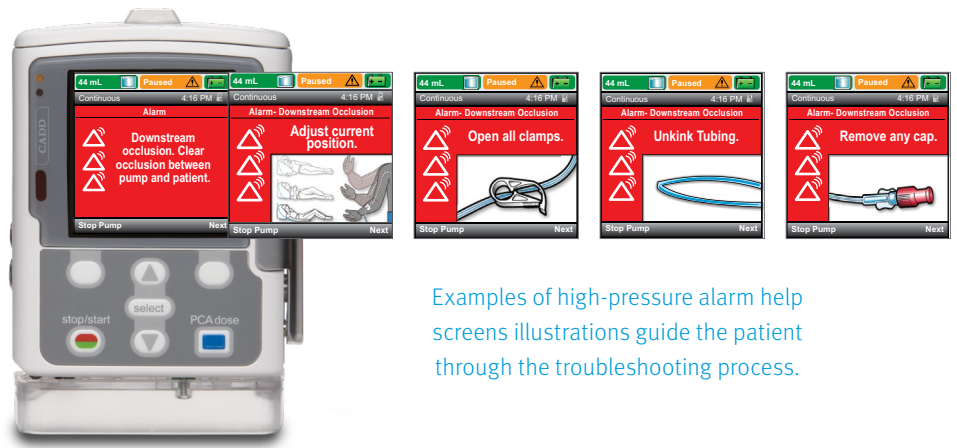


Examples of high-pressure alarm help screens illustrations guide the patient through the troubleshooting process.

Differentiated alarms


Differentiated alarms by colour and sound with high (red), medium (amber) and low (blue) priority

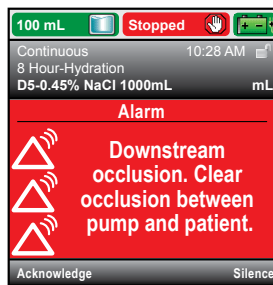

High-priority alarm

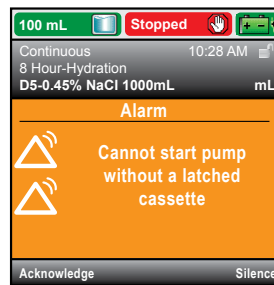


Medium-priority alarm

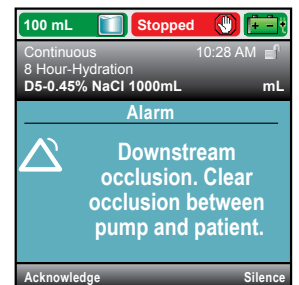


Low-priority alarm

On-pump reports

On-pump reports provide trending data and help with patient assessments.

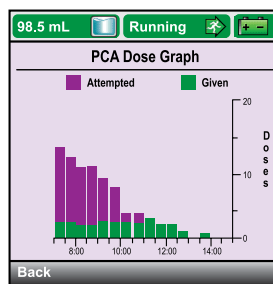

Dosing report screen

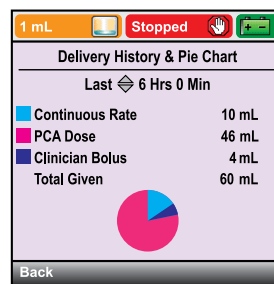

Delivery history report screen

CADD-Solis VIP Ambulatory Infusion System

Easier for Patients and Nurses

Immediate Pump Status

Colour-coded status bar supports visual indication of pump status.

Quickly View Drug Protocols

Protocol bar supports easy viewing of selected therapy, qualifier and drug.

Intuitive Programming

Get step-by-step instructions on colour-coded screens.

Easy Navigation

Simplify searching with soft-key interface.



See Pump Lock Status

Lock status indicator supports clinician confidence.

Clear Display

Features easy-to-see infusion delivery parameters with unit of measure.

Easy Disposable Attachment

Convenient Dosing

PCA dose key supports PCA dosing while ambulatory.

Support your patients' independence today.

Call +44 (0)845 850 0445 or visit www.icumed.com to learn more.

The product complies with current legislation and has the corresponding CE marking (2797).
For additional information, warnings and / or safety precautions, refer to the manufacturer's user manual.

icumedical
human connections