




Product Reference Guide

Flexible IV Container


Adding a Medication

- › The flexible plastic container is fabricated from a specially formulated polyvinylchloride.
- › Water can permeate from inside the container into the overwrap but not in amounts sufficient to affect the solution significantly.
- › Additives may be incompatible. Consult with pharmacist, if available.

To Add a Medication: (Use aseptic technique)

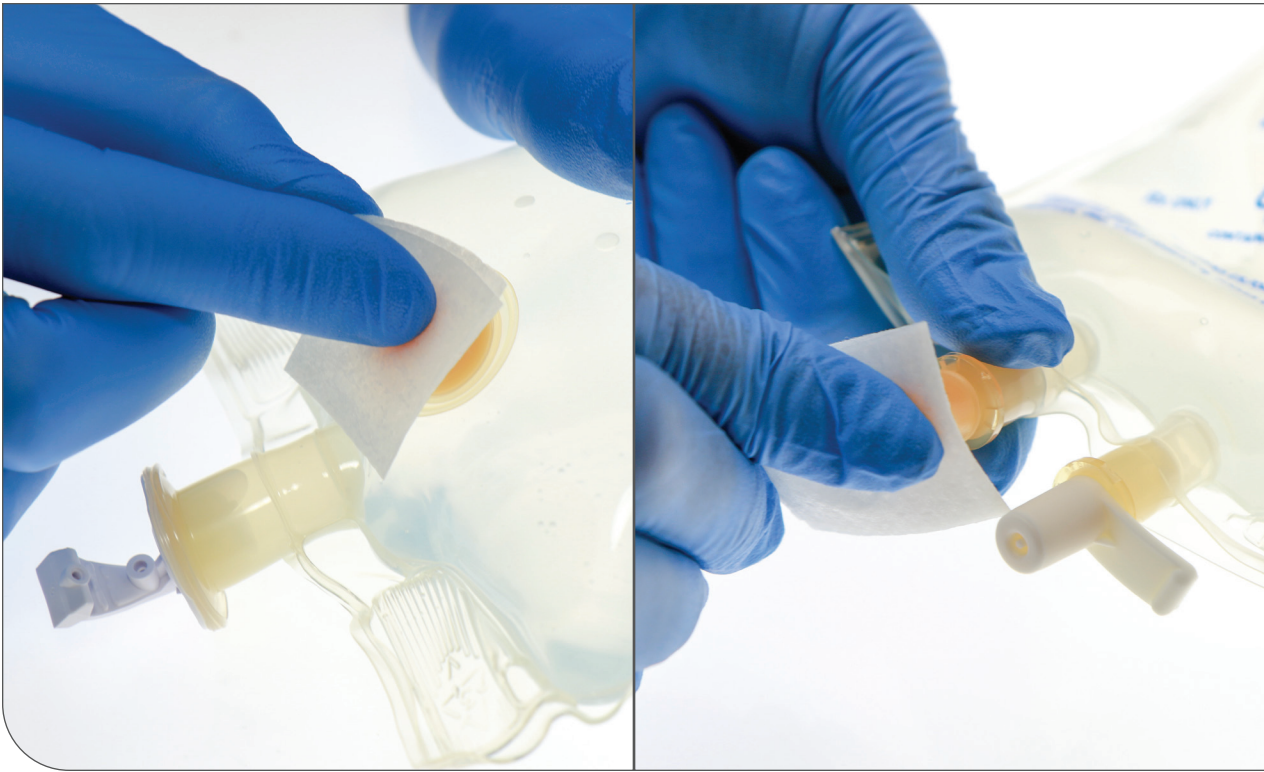

1. Prepare additive port.

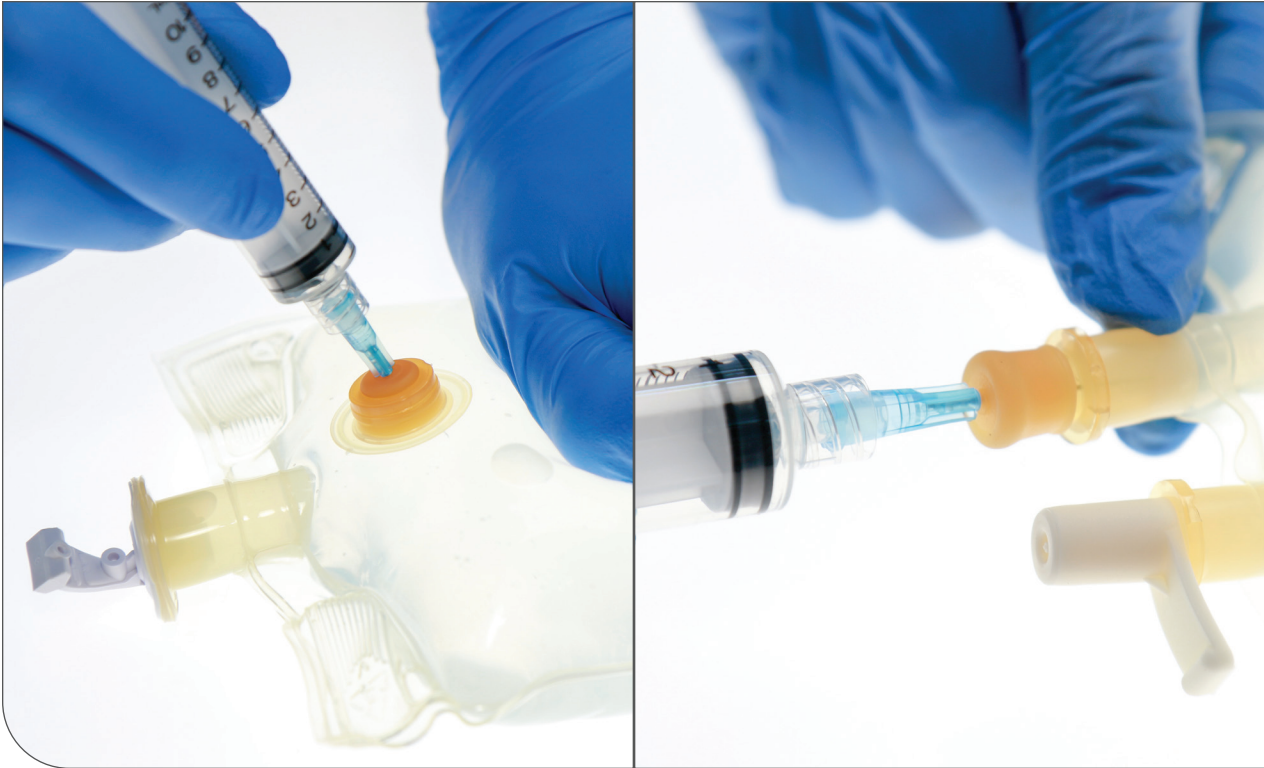

2. Using aseptic technique and an additive delivery needle of appropriate length, puncture resealable additive port at the target area, inner diaphragm, and inject.
3. Withdraw needle after injecting the medication.



4. The additive port may be protected by covering with an additive cap.
5. Mix container contents thoroughly. Do not store.