

3000 mL

List No. 7927-20

— 2750 —  
 — 2500 —  
 — 2250 —  
 — 2000 —  
 — 1750 —

**EMPTY  
CONTAINER**  
with Attached  
Y-Transfer Set,  
Convertible Pins

2750 —  
 2500 —  
 2250 —  
 2000 —  
 1750 —

*SINGLE-USE  
I.V. CONTAINER  
FOR ADMIXING*

*COMPATIBLE MEDICATIONS.*

— 1250 —  
 — 1000 —

FLUID PATH AND AREAS BENEATH UNDISTURBED PROTECTIVE SET COVERS ARE STERILE AND NONPYROGENIC IN INTACT UNIT PACKAGE. USE ASEPTIC TECHNIQUE. MIX CONTAINER CONTENTS THOROUGHLY. ALL GRADUATIONS ARE APPROXIMATE. LABEL APPROPRIATELY. BEFORE ADMINISTRATION USE, CHECK FOR MINUTE LEAKS BY SQUEEZING CONTAINER FIRMLY. DO NOT USE IF CONTAINER IS DAMAGED. ADHERE TO STORAGE REQUIREMENTS OF ADDED MEDICATIONS.

1250 —  
 1000 —

— 750 —

WARNINGS: TO AVOID CIRCULATORY OVERLOAD, USE ONLY WITH AN ELECTRONIC FLOW CONTROL DEVICE EQUIPPED WITH A FLOW CONTROL ALARM SYSTEM. MUST NOT BE USED IN SERIES CONNECTIONS.

750 —

— 500 —

MADE IN COSTA RICA.

PRODUCT INQUIRIES SHOULD BE DIRECTED TO ICU MEDICAL, INC., LAKE FOREST, ILLINOIS, 60045, USA

500 —

Rx ONLY

— 250 —



Not made with natural rubber latex



CONTAINS DEHP  
Recyclable as Type-3 Plastic

250 —

— 100 — IM-4240 (06/2017) **icumedical** 100 —