



Product Reference Guide

MicroClave™ Connector Extension Sets

1. Open the set



- › Using an aseptic technique, open the package and tighten the connections where applicable.

2. Prime and attach the set



- › Prime the connector and set by attaching the priming device to the connector and inverting to expel air.
- › Repeat for multiple extensions as necessary.
- › If the set has multiple extensions, prime each extension per facility protocol.

3. Attach the set



- › Attach the male luer of the MicroClave connector or extension set to the end of the catheter hub by pulling the spin collar back to expose the male luer.

4. Insert into the catheter



- › Insert the end of the male luer into the catheter hub, pushing straight into the connector and twisting a 1/4 turn to the right until a friction fit is achieved.

5. Tighten the spin collar




- › Move the spin collar back into place and twist to lock.
- › Grasp the rigid portion behind the spin collar with one hand and the spin collar with the other and continue to tighten the spin collar onto the catheter threads.

6. Disinfect



- › Swab the top of the MicroClave connector per facility protocol (use appropriate scrubbing and dry times) and allow to dry.

7. Access

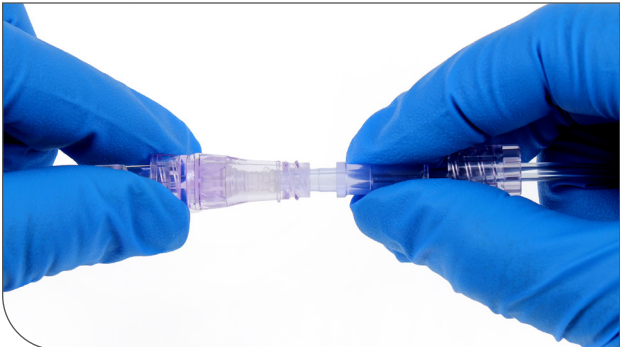

- › The MicroClave connector may be accessed with any spin luer or slip luer connection.
- › Do not use with needles or blunt cannulas. For sets/devices containing a rotating collar, ensure the luer connector is fully engaged prior to tightening the collar.
- › Do not overtighten; this may cause damage.
- › To access with primary tubing: Swab the top of MicroClave connector per facility protocol, pull back the spin collar, insert the end of the slip



- › luer, push and twist 1/4 turn to the right until secure, and lower and tighten the spin collar.
- › To draw blood with a vacutainer: Swab the top of MicroClave connector, withdraw and discard the waste with the syringe, swab the top of the MicroClave, insert the end of the slip luer (push and twist 1/4 turn to the right until secure) or male luer, and draw blood. Be sure to twist the adapter in the same direction so the needle isn't exposed when removing. Swab the top of the MicroClave once removed.

8. Flush



- › Flush the MicroClave connector according to facility protocol.



ANTT® Evaluated Technology promotes safe and effective Aseptic Non-Touch Technique principles by design.

