Supercath 5

Safety I.V.Catheter INSTRUCTIONS FOR USE

Read this user manual carefully before using the device.

ONLY * Federal law (USA) restricts this device to sale by or on order of a physician.

INDICATIONS

The SUPERCATH 5 is intended to access a vein or artery and to administer fluids and/or blood. The SUPERCATH 5 is designed for single use, and is designed to minimize inadvertent needlesticks and to reduce accidental needlesticks. The 18-22 gauge catheters may be used with power injectors at a maximum pressure of 300 psi.


DESCRIPTION

The SUPERCATH 5 is designed to minimize inadvertent needlesticks and to reduce accidental needlesticks. The SUPERCATH 5 catheter hub has a built-in check valve which together with the healthcare professional's finger pressure on the blood vessel, assists to reduce blood flashback when the metallic introducer needle is withdrawn following blood vessel puncture. The built-in check valve is not intended to stop bleeding completely. Pressing the button on the needle hub activates the coiled spring in the hub, retracting the metallic introducer needle into the needle hub.

The SUPERCATH 5 introducer needle has a side-notch to provide rapid visual confirmation of vessel entry. When the introducer needle is inserted into the vein, blood flows up and through the side-notch and returns down along the inside of the catheter tube. The SUPERCATH 5 is available in 14G, 16G, 18G, 20G, 22G, 24G and 26G.


Gauge	Color code	O.D. (mm)	I.D. (mm)	Length	
				(in)	(mm)
26G	Violet	0.6	0.4	0.56	14
26G	Violet	0.6	0.4	¾	19
24G	Yellow	0.7	0.4	0.56	14
24G	Yellow	0.7	0.4	¾	19
22G	Deep Blue	0.9	0.6	1	25
22G	Deep Blue	0.9	0.6	1¼	31
20G	Pink	1.1	0.7	1	25
20G	Pink	1.1	0.7	1¼	31
20G	Pink	1.1	0.7	2	50
18G	Deep Green	1.3	0.9	1¼	31
18G	Deep Green	1.3	0.9	2	50
16G	Medium Grey	1.7	1.3	1¼	31
14G	Orange	2.1	1.6	1¼	31

< Without Wing >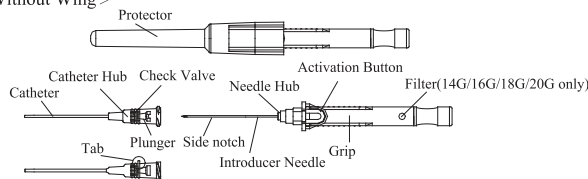

- * Not made with natural rubber latex.
- * This product does not contain or have presence of Bis(2-ethylhexyl)phthalate (DEHP).

< With Wing >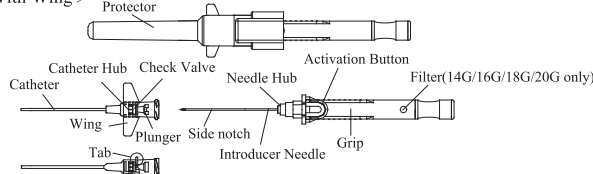

- * Not made with natural rubber latex.
- * Bis(2-ethylhexyl)phthalate (DEHP) materials are not in the fluid pathway; only contained in the catheter wing.

WARNING

- Do not resterilize, reuse or re-process. FOR SINGLE USE ONLY. Resterilization, reuse, or reprocessing can cause contamination of the device, which may lead to infection of the patient or cross infection.

PRECAUTIONS

- Use caution when handling the SUPERCATH 5 and maintain proper aseptic technique during insertion, withdrawal, and disposal as well as when the catheter is in patient use.
- Do not move the introducer needle forwards or backward in the catheter before use or during puncture.
- Do not reinsert a partially or completely withdrawn needle.
- Please refer to your facility protocols for appropriate use intervals.
- If you fail to secure blood vessel access, use a new catheter to puncture the vessel again.
- Take care not to press the activation button accidentally before use or during puncture.
- Do not use if the package is wet, opened, dirty, or damaged.
- Use immediately after package is opened. Discard properly after use.
- Perform all manipulations aseptically.
- Do not use sharps such as scissors near the catheter.
- The SUPERCATH 5 should only be used by healthcare professionals familiar with catheterization techniques.
- Do not use for purposes other than those intended.
- Store the SUPERCATH 5 in a dry (humidity below 75%), dark (avoid direct sunlight), room-temperature (60-80 °F(15.6-26.6 °C)) cabinet or closet.
- Wear proper gloves during the procedure.
- Do not bend the needle while using the product.
- Carefully evaluate and select the appropriate size catheter.

- The maximum flow rate and maximum pressure setting for each catheter size are listed in the table below. The maximum flow rate may vary due to differences in injector characteristics, injectate viscosity and temperature, and other variables.

Gauge	Length		Maximum Flow Rate (mL/sec)	Maximum pressure (psi)
	(in)	(mm)		
22G	1	25	7	300
22G	1¼	31	7	300
20G	1	25	9	300
20G	1¼	31	9	300
20G	2	50	9	300
18G	1¼	31	12	300
18G	2	50	12	300

MRI SAFETY INFORMATION


MR MR Safe

The "Catheter" is composed of materials that are electrically nonconductive, nonmetallic, and nonmagnetic. Non-clinical testing demonstrated that Supercath 5 Safety IV Catheters are MR Safe and pose no known hazards resulting from exposure to an MR environment. The material of the "Introducer Needle" is not Magnetic Resonance (MR) compatible. Follow facility policies and guidelines when utilizing the IV Catheter within the MRI suite.

INSTRUCTIONS FOR USE

- Open the package.
 - Hold the safety cover, and remove the protector.
Take care not to damage the needle tip.
 - Check the condition of the introducer needle tip. Prior to patient insertion, separate the catheter from the introducer needle by twisting and reseat the catheter hub back into position with the advancing tab facing upward in alignment with the push button.
Avoid touching the catheter or the metallic introducer needle while performing this separation. Separation of the catheter from the introducer needle will help ensure proper catheter advancement and needle retraction during venipuncture.
- [Note]** The built-in check valve together with the healthcare professional's finger pressure, assists to reduce blood flashback when the introducer needle is withdrawn following blood vessel puncture. The built-in check valve is not intended to stop bleeding. Observe the catheter hub carefully when removing the introducer needle. If any sign of blood leakage is found, apply appropriate compression to achieve hemostasis.
- [Note]** The check valve is intended for single use. Since it does not stop bleeding following removal of an infusion or transfusion set, apply pressure when disconnecting the male connector from the catheter hub.
- [Note]** Firmly press the activation button to retract the needle into the safety cover.
- [Note]** Bending the needle before or during use may affect proper needle retraction and proper check valve functionality.
- Promptly dispose of the SUPERCATH 5 into a sharps container immediately after use and follow facility protocol.
 - Securely connect the infusion or transfusion set to the catheter hub.
[Note] Take care not to introduce air bubbles during connection.
[Note] Before starting the infusion or transfusion, verify that the plunger has pierced the check valve and that sufficient flow has been established.
 - Secure the catheter and the infusion or transfusion tubing line with sterile gauze and tape or a transparent semi-permeable dressing. These dressings should be applied only to the catheter adapter and should not be placed directly on the catheter-skin junction site.

Explanation of symbols

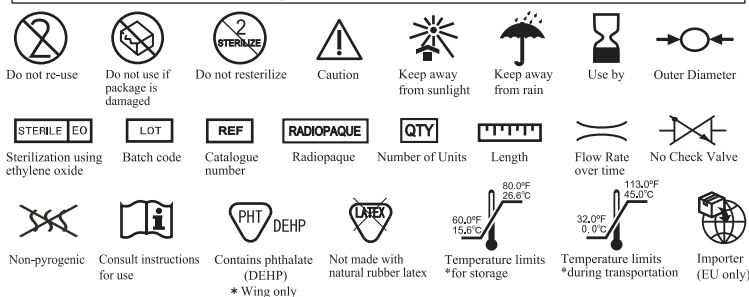Manufacturer: Togo Medikit Co., Ltd.
17148-6, Aza Kamekawa, Oaza Hichiya,
Hyuga City, Miyazaki prefecture, Japan
883-0062

EMERGO EUROPE
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

Distributed by: icumedical
www.ICUMED.com

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Fabricant: Togo Medikit Co., Ltd.