



Cleo™ 90 Infusion Set

All-in-one subcutaneous infusion set

The advantages of the Cleo 90 infusion set are clear

All-in-one design

The preassembled Cleo 90 infusion set features a unique, proprietary housing that includes an introducer and cannula to make insertions simple and allow you to control insertion speed.

Enclosed needle safety

The Cleo 90 infusion set is a system with its own needle protection device. The enclosed needle retracts automatically

into the housing after insertion, reducing the risk of accidental needlestick injuries.

Hypoallergenic dressing

The small, transparent adhesive dressing is designed to breathe and stay in place with minimal tape. Its proprietary release tab makes removal easy too.

Flexible 360° connection

The Flex Attach™ 360° site tubing

connection provides 360 on-site rotation to allow for greater flexibility.

Easy to train

The Cleo 90 infusion set can be inserted with a single motion using one hand. Designed with a standard luer connection, Cleo can be used with any pump that has a luer fitting.


As easy as twist, insert, and connect



Twist cap to remove.



Insert the cannula. Press device firmly until you hear a click, then maintain for at least 5 seconds. Gently pull off the applicator.

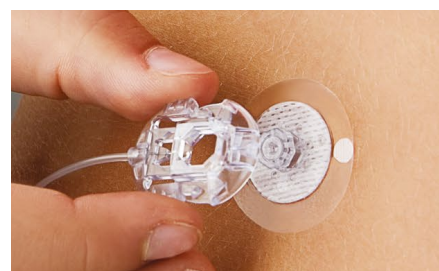

Connect the tubing set.

A complete range of subcutaneous infusion sets adapted to each person's needs



Standard luer lock connection

Cleo 90 Infusion Set — straight angle catheter

Item reference	Cannula length (mm)	Tubing length
21-7220-24	6	61 cm (24")
21-7221-24	6	79 cm (31")
21-7222-24	6	107 cm (42")
21-7230-24	9	61 cm (24")
21-7231-24	9	79 cm (31")
21-7232-24	9	107 cm (42")

Supplied sterile in cases of 10 units
28G Introducer Needle and 25G Cannula

The product complies with current legislation and has the corresponding CE marking (2797).
For additional information, warnings and/or safety precautions, refer to the manufacturer's Instructions for Use.

icumedical
human connections