



Cleo™ 90 Infusion Set

Adhesion Guide

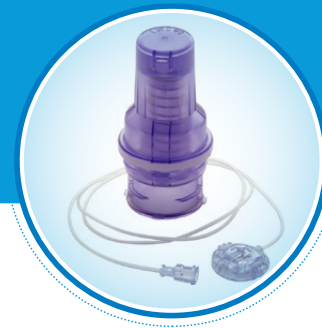

Below are some tips to assist you during the initial application of the Cleo 90 adhesive site. Prepare your insertion site by avoiding the following:

- › Moisture of any kind
- › Hair; remove hair around the insertion site to ensure the adhesive sticks to skin if needed
- › Oily residue from soap or lotion

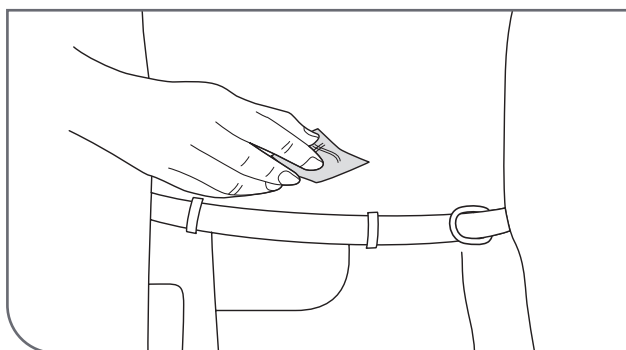


1. Prepare site; allow to dry.

Allow your skin to completely air dry. You may also want to consider using a product such as the following for enhanced adhesion.*


- › SKIN TAC® (Torbot)
- › Cavilon® No Sting Barrier Film (3M)
- › NO-STING SKIN-PREP® (Smith & Nephew)
- › Tegaderm® Transparent Dressing (3M) – to be placed under the infusion site before insertion

Remember to allow for adequate drying time as applicable.

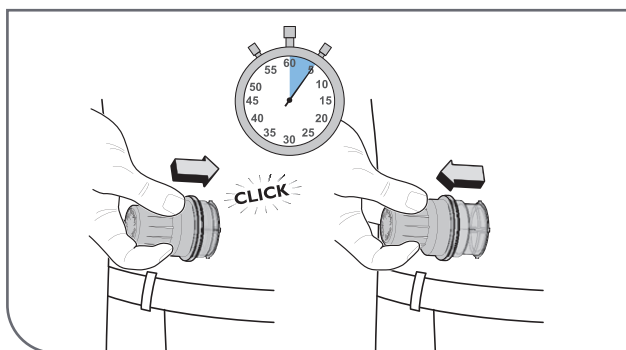

2. Press firmly until you hear a click; hold for 5 seconds; pull straight off.

Once you hear a click, hold down for at least 5 seconds. With one hand, gently pull off the applicator and with the other hand, simultaneously hold down a corner of the adhesive.

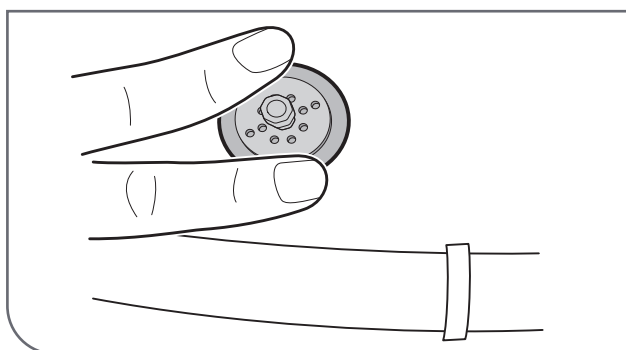

3. Press to secure adhesive.

Press down along the edges of the adhesive to make sure it adheres to your skin.

SKIN TAC is a registered trademark of Trobot Group, Inc.

Cavilon and Tegaderm are registered trademarks of Solvatum Intellectual Properties Company

SKIN-PREP is a registered trademark of Smith & Nephew, Inc.

*This does not constitute an endorsement of the manufacturer or the product quality. ICU Medical is not responsible for the completeness or accuracy of product information. Product availability may vary by country and region. The manufacturer's Instructions for Use of each product should be followed.

icumedical
human connections