


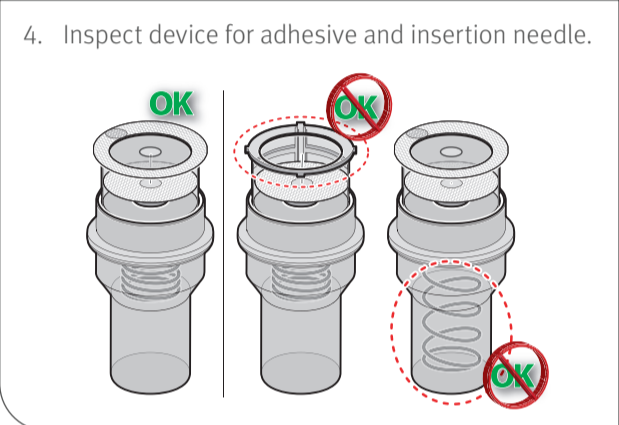
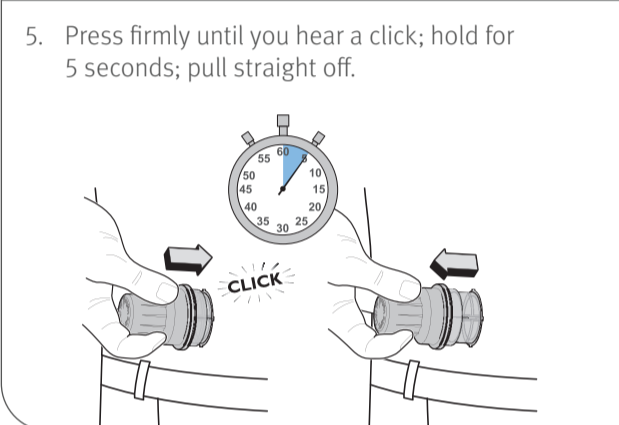
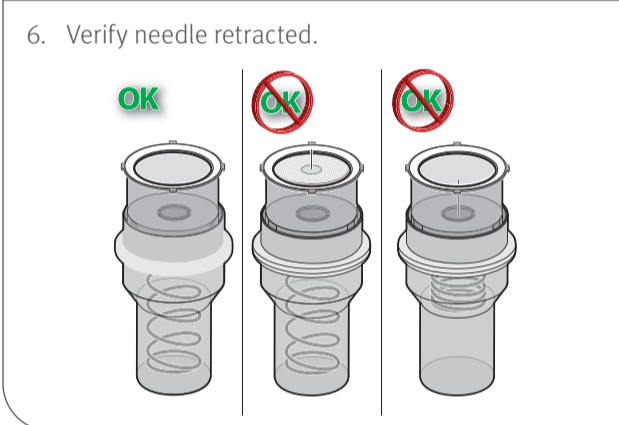
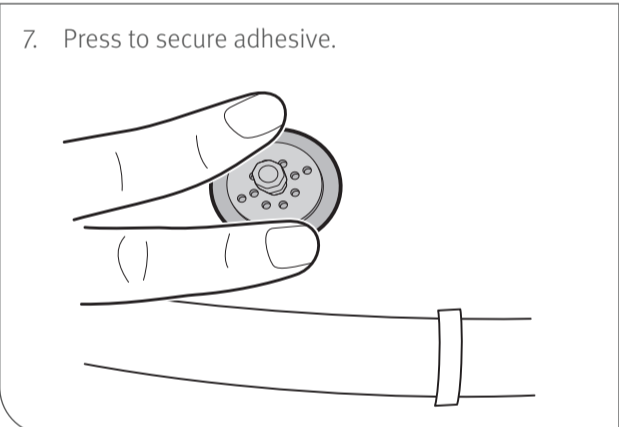
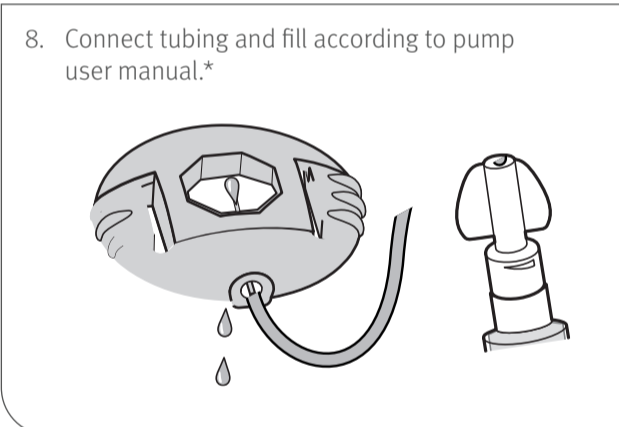
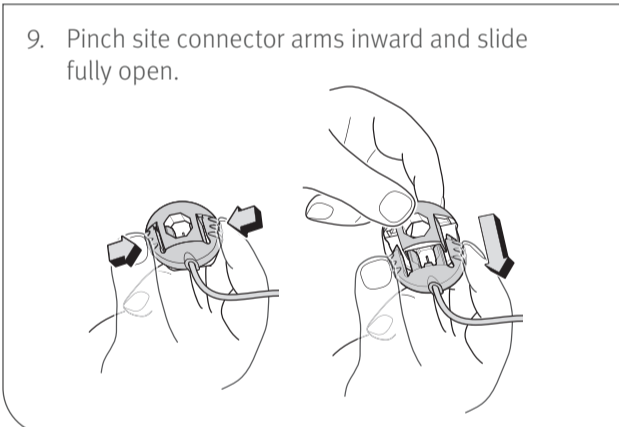
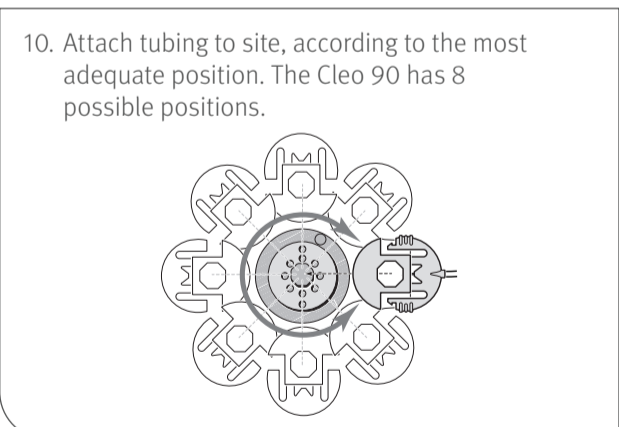
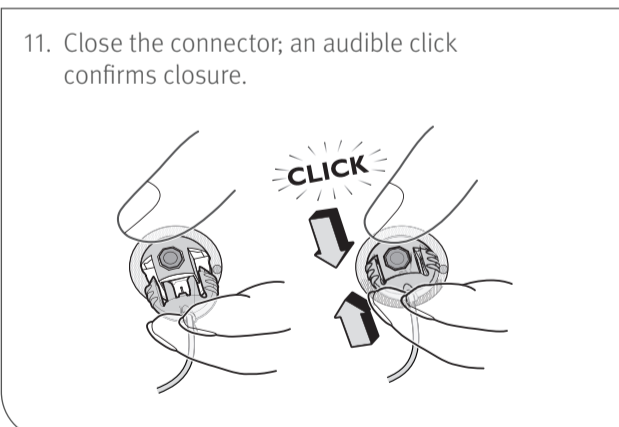


Cleo™ 90 Infusion Set Training Guide




Please see Instructions for Use for a complete list of the indications, contraindications, warnings and precautions.

Insert site and connect tubing set



1. Wash and dry hands thoroughly.
 
2. Prepare site; allow to dry.
 
3. Twist cap to remove.
 
4. Inspect device for adhesive and insertion needle.
 
5. Press firmly until you hear a click; hold for 5 seconds; pull straight off.
 
6. Verify needle retracted.
 
7. Press to secure adhesive.
 
8. Connect tubing and fill according to pump user manual.*
 
9. Pinch site connector arms inward and slide fully open.
 
10. Attach tubing to site, according to the most adequate position. The Cleo 90 has 8 possible positions.
 
11. Close the connector; an audible click confirms closure.
 
12. Fill cannula with medication.
 - › The 6 mm cannula has 0.002 mL priming volume.
 - › The 9 mm cannula has 0.003 mL priming volume.

Disconnection

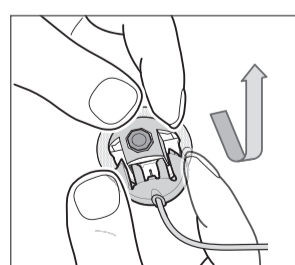

Open the connector by pinching the site connector arms inward and sliding fully open.

Withdrawal of the injection site

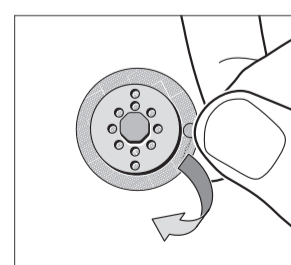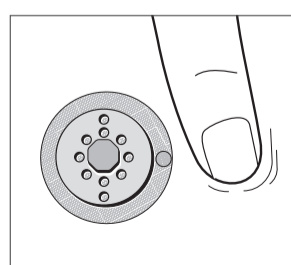

Changing the Cleo 90 catheter

Change the catheter per your facility policy and procedures.

* Warning: Never fill the tubing with medication or insulin while the infusion set is connected to your body, or you could deliver an unintended bolus of medication.